<b>DE</b> Dampfvernebler Gebrauchsanweisung.....	2	<b>IT</b> Nebulizzatore a vapore Istruzioni per l'uso .....	34
<b>EN</b> Steam vaporiser Instructions for use .....	10	<b>TR</b> Buharlı atomizer Kullanım kılavuzu.....	42
<b>FR</b> Nébuliseur à vapeur Mode d'emploi .....	18	<b>RU</b> Паровой ингалятор Инструкция по применению .....	50
<b>ES</b> Nebulizador Instrucciones de uso .....	26	<b>PL</b> Nebulizator parowy Instrukcja obsługi .....	58

## Inhalt

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## Contents

1. Getting to know your device .....	10
2. Signs and symbols .....	10
3. Warnings and safety notes .....	11
4. Description of the device and accessories .....	13
5. Operating the steam vaporiser .....	13
6. Cleaning and disinfection .....	14
7. Disposal .....	16
8. Troubleshooting .....	16
9. Technical specifications .....	16
10. Replacement parts and wearing parts .....	16
11. Warranty/service.....	16

## Included in delivery

- Steam vaporiser
- Mask
- These instructions for use

## 1. Getting to know your device

### Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use. Be sure to make them accessible to other users and observe the information they contain.

With kind regards,  
Your Beurer team

### Intended use

This device is specifically designed to generate steam using vaporisation, in order to humidify the ambient air. You can also use it with supplements, such as essential oils.

Once it has been properly prepared, the device can be used again. Preparation involves replacing the mask and disinfecting the surface of the device using

a conventional disinfectant. Please note that the mask should be replaced if the device is used by more than one person.

We recommend that you replace the mask after one year.

## 2. Signs and symbols

The following symbols are used in this manual.

 <b>Warning</b>	Warning notice indicating a risk of injury or damage to health.
 <b>Important</b>	Safety note indicating possible damage to the device/ accessory.
 <b>Note</b>	Note on important information.
	Observe the instructions for use
	Protection class 2 device
	Manufacturer
I	On
O	Off
<b>IP 21</b>	Protected against foreign objects equal to or greater than 12.5 mm in diameter and against vertically falling drops of water
	This product satisfies the requirements of the applicable European and national directives.
	Dispose of packaging in an environmentally friendly manner
	Batch designation

### 3. Warnings and safety notes

#### Warning

- Before use, ensure that there is no visible damage to the device or accessories. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- In the event of device faults, please see Chapter “8. Troubleshooting”.
- This is not a medical device. Always consult your doctor if you require medical treatment.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- Please observe the general hygiene measures when using the steam vaporiser.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- Do not position your face too close to the mask, otherwise the hot steam could cause burn injuries. Close your eyes during the treatment and protect heat-sensitive zones.
- Never open the device (remove the cover, unscrew the lid) while it is in operation. Allow the device to cool down first. Otherwise, you may get burnt.
- Do not operate the device in the presence of flammable gases or if there is a high oxygen concentration.
- Never leave the device unattended while in use.
- The device must be switched off and the plug pulled out before every cleaning and/or maintenance procedure.
- Keep packaging material away from children (risk of suffocation).
- To avoid the risk of entanglement and strangulation, store the cable out of the reach of small children.
- Do not use any additional parts that are not recommended by the manufacturer.
- The device must only be connected to the mains voltage that is specified on the type plate.
- Never submerge the device in water and do not use it in the bathroom. Under no circumstances may liquid enter the device.
- Protect the device from heavy impact.
- Never touch the mains cable with wet hands, as you could get an electric shock.
- Do not pull the mains plug out of the socket using the mains cable.
- Do not crush or bend the mains cable, pull it over sharp-edged objects or leave it dangling down, and protect it from sources of heat.
- We recommend that the power cable is completely unrolled to avoid dangerous overheating.
- If the mains connection cable of this device is damaged, it must be replaced by the manufacturer, its customer service department or a similarly qualified person to avoid any risks.
- If the device is opened, there is a risk of electric shock. Disconnection from the power supply network is only guaranteed if the mains plug is unplugged.
- If the device has been dropped, exposed to high levels of moisture or suffered any other damage, it must no longer be used. If in doubt, contact Customer Services or the retailer.
- The steam vaporiser may only be operated with the appropriate accessories.

#### Important

- Power cuts, sudden interference or other unfavourable conditions could lead to the device becoming inoperable. For this reason, we recommend that you have a spare device at your disposal.
- Never use the device with extension cables.
- Never store the device or the power cable near to sources of heat.
- Do not use the device in a room in which a spray has previously been used. Air the room before carrying out the treatment.
- Never insert anything into the vents.
- Never block the vents.
- Never use the device when the aroma container is empty.

- Never use the device when the water heater tank is empty.
- For hygiene reasons, it is essential that every user uses their own accessories.
- Always disconnect the mains plug from the mains socket after use.
- Store the device in a location protected against climatic influences. The device must be stored in the environmental conditions specified.

## PROTECTION

- The device has a protection fuse. This may only be replaced by authorised specialist personnel.
- The device also has a thermal fuse, which switches the steam vaporiser off if it overheats. In the event of overheating, the light on the ON/OFF switch turns off automatically. If this happens, proceed as follows:
  1. Switch off the device first.
  2. Pull the mains plug out of the socket.
  3. Wait for at least 30 minutes until the device has cooled down completely.
  4. Switch the device back on and check whether it is working properly again. If it does, you can continue using the device again. If it does not, please contact Customer Services or an authorised retailer.

## General notes

### Important

- The device is suitable only:
  - for use on people,
  - for the purpose for which it was developed and as specified in these instructions for use.
- **Any form of improper use can be dangerous.**
- In the event of an acute emergency, providing first aid must take top priority.
- Only use distilled water and aroma oils. Other liquids may potentially cause a fault in the steam vaporiser.
- This device is not intended for commercial or clinical use; it is designed exclusively for self-treatment in a private home.

## Prior to initial use

### Important

- Remove all packaging material before using the device.
- Protect the device against dust, dirt and humidity and never cover the device while it is in use.
- Do not operate the device in a very dusty area.
- Switch the device off immediately if it is faulty or not working properly.
- The manufacturer is not liable for damage resulting from improper or incorrect use.


## Repairs

### Note

- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply with this instruction will void the warranty.
- The device is maintenance-free.
- For repairs, please contact Customer Services or an authorised retailer.

## 4. Description of the device and accessories

### Overview of the steam vaporiser

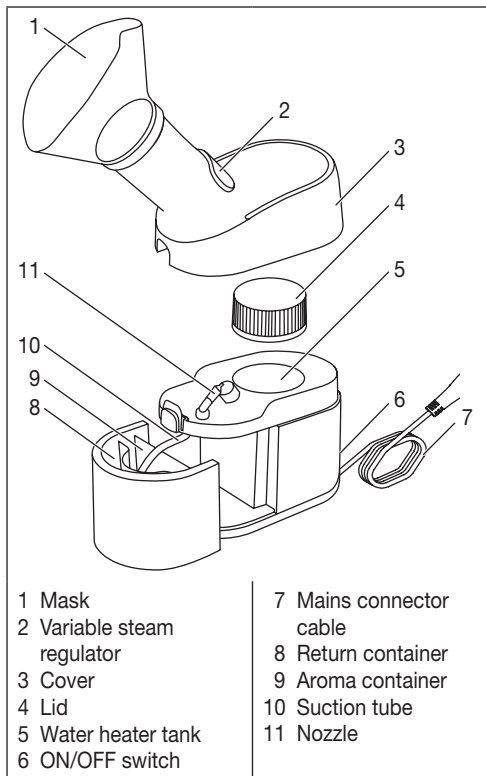## 5. Operating the steam vaporiser

### **i** Note

Follow the instructions in the Chapter “Cleaning and disinfection” before using the device for the first time or if it has not been used for an extended period.

1. Remove the device from the packaging. Ensure that there is a mains socket close to the set-up area.

2. Place the device on a firm, even surface and pull the cover up and off by pressing the button above the aroma container/return container.

3. Remove the sticker with the warning notice from the cover. Twist the lid anti-clockwise. Observe the marking on the lid when you do this. Remove the lid.


4. Fill the water heater tank with approx. 25 ml of water (do not fill the water heater tank above the “MAX” marking). Always use water to fill the water heater tank and never use any other liquids.

5. Refit the lid and close it tightly by turning it clockwise. Observe the marking on the lid when you do this.

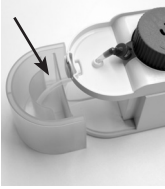6. Replace the cover from above.

<p>7. Pull out the aroma container/return container. Fill the aroma container with the liquid that you want to atomise (e.g. salt-water solution). Never fill the aroma container above the “MAX” marking (60ml).</p>	
<p>8. Re-insert the aroma container/return container into the device. Make sure that the suction tube touches the bottom of the aroma container.</p>	
<p>9. Push the variable steam regulator down to produce smaller steam particles, and push it up for larger ones.</p>	
<p>10. Make sure that the ON/OFF switch is in the “OFF” position. Arrange the mains cable in such a way that no-one will trip over it. Insert the mains plug into a suitable socket. The device must only be connected to the mains voltage that is specified on the type plate.</p>	
<p>11. Move the ON/OFF switch to the “ON” position.</p>	

<p>12. After approx. 2 minutes, the device will start to produce steam. During use, sit in an upright and relaxed position at a table and not in an arm-chair, so as to avoid compressing the airways and therefore impairing effectiveness. Since each person’s temperature sensitivity is different, we recommend starting use with sufficient distance. Do not hold your face directly against the mask. Gradually reduce the distance. Breathe in deeply. Ensure that steam does not escape from the mask and into the eyes. As a result, it is recommended that you close your eyes during treatment. As soon as the production of steam starts to diminish (maximum duration approx. 10 minutes), turn the ON/OFF switch to the “OFF” position.</p>
<p>13. After each use, pull the mains plug out of the socket. Allow the device to cool down completely (approximately 10 minutes).</p>
<p>14. Clean the device and accessories after every use as described in the “Cleaning and disinfection” chapter.</p>

## 6. Cleaning and disinfection

### Warning

Adhere to the following hygiene instructions to avoid health risks. The device must be switched off, disconnected from the mains and allowed to cool down each time before cleaning.


#### The device must be cleaned after every use

1. Make sure that the device has cooled down (approx. 10 minutes), the ON/OFF switch is in the “OFF” position and the device has been disconnected from the power supply.
2. Pull the cover of the device up and off by pressing on the button above the aroma container/return container.

3. Twist the lid anti-clockwise. Observe the marking on the lid when you do this. Remove the lid. Empty any remaining water from the water heater tank. Then dry it with a clean cloth or let it air-dry completely. Residual moisture or wetness represents an increased risk of bacterial growth.

4. Pull out the aroma container/return container.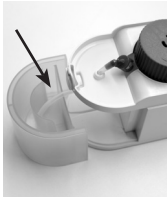

5. Clean the aroma container/return container, the cover, the lid, the extension tube and the mask with warm water and a mild cleaning agent. Clean these parts thoroughly and allow them to dry completely. Residual moisture or wetness represents an increased risk of bacterial growth.
6. Once the parts are completely dry, you can reassemble the device.
7. Store the device in a cool, dry place.

### Cleaning the main unit

- Never submerge the main unit in water.
- Clean the main unit using a soft, slightly damp cloth and a mild cleaning agent. Then wipe dry with a soft, lint-free cloth. Do not use any chemical or abrasive cleaning agents.

### Note


Any other cleaning method or type of cleaning product could damage the device. Failure to comply with this instruction will void the warranty.

### Cleaning the nozzle

### Note

Periodically, the nozzle that directs the steam into extension tube may become blocked. When the device starts to produce significantly less steam, or no steam at all, you need to clean the nozzle.

1. Pull the cover up and off.
2. Turn the nozzle in a clockwise direction. Pull the nozzle up and off.
3. Insert a thin object (for example a needle) into the nozzle in order to clean the air duct.

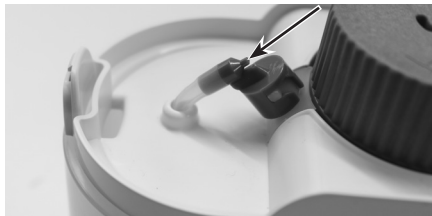4. To reattach the nozzle, align the grips of the nozzle with the recesses on the nozzle head. Turn the nozzle anticlockwise, until you feel the grips of the nozzle click into place.

### Disinfection

The mask can be disinfected using boiling water.

- Remove the mask from extension tube.
- Place the mask into boiling water for 5 minutes.
- Then dry the mask carefully using a soft cloth.
- Re-attach the mask to the extension tube once it has completely dried.

### Storage

- Do not store in damp conditions (such as in a bathroom) and do not transport with any damp items.
- When storing and transporting, protect from prolonged direct sunlight.

- Store the device in a dry place, ideally in the original packaging.

## 7. Disposal

For environmental reasons, do not dispose of the device in the household waste.

Please dispose of the device in accordance with EC Directive – **WEEE** (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

## 8. Troubleshooting

Problem/question	Possible cause/remedy
The ON/OFF switch has been switched on, but the device is not working.	<ol style="list-style-type: none"> <li>1. Check the power connection to the socket and check whether the mains plug has been plugged in correctly.</li> <li>2. The device has a thermal fuse. Following switch-off, the mains plug must be pulled out and a waiting period observed (see the section on fuses in Chapter 3 „Warnings and safety notes“).</li> </ol>
The device is switched on but produces no or hardly any steam.	1. Check that there is water in the container of the water heater tank.
	2. Check that there is liquid in the aroma container.
	3. Make sure that the suction tube and other components have been attached properly.
	4. Check the nozzle for blockages. Clean the nozzle if necessary (see “Cleaning the nozzle” section in the “Cleaning and disinfection” chapter)
Does everyone need a separate mask?	Yes, this is absolutely essential for hygiene reasons.

## 9. Technical specifications

Model	SI 40
Dimensions (W x H x D)	265 x 260 x 96 mm
Weight	620 g
Filling volume of internal water heater	Max. 25 ml
Aroma container filling volume	Max. 60 ml
Mains connection	220 – 240 V~; 50 – 60 Hz
Steam temperature	Approx. 43°C
Water boiling time	Approx. 2 minutes
Maximum operation time	Approx. 10 minutes
Operating conditions	Temperature: +10°C to +40°C Relative humidity: 10% to 95% Ambient pressure: 700 hPa to 1060 hPa
Storage and transportation conditions	Temperature: 0°C to +60°C Relative humidity: 10% to 95% Ambient pressure: 500 hPa to 1060 hPa

Subject to technical changes.

Please contact the specified service address to obtain further information, such as the CE Declaration of Conformity.

## 10. Replacement parts and wearing parts

Designation	Material	REF
Mask	PVC	601.34

## 11. Warranty / service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer. The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached "International Service" list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions

of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;

- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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