GET READY!

babyono

<b>GB</b>	COMPACT electric breast pump with 5 modes	2
<b>PL</b>	Laktator elektryczny COMPACT z 5 trybami pracy	6
<b>BG</b>	Електрическа помпа за кърма COMPACT с 5 режима на работа	10
<b>CZ</b>	Elektrická odsávačka mateřského mléka COMPACT s 5 pracovními režimy	14
<b>DE</b>	Milchpumpe COMPACT mit 5 Arbeitsmodi	18
<b>EE</b>	5 töörežiimiga elektriline rinnapump COMPACT	22
<b>ES</b>	Extractor eléctrico de leche materna COMPACT con 5 modos	26
<b>FR</b>	Tire-lait électrique COMPACT 5 modes	30
<b>GR</b>	Ηλεκτρική συσκευή άντλησης μητρικού γάλακτος COMPACT με 5 λειτουργίες	34
<b>HR</b>	Električni laktator COMPACT s 5 načina rada	38
<b>HU</b>	COMPACT elektromos mellészívó 5 különböző üzemmóddal	42
<b>IT</b>	Tiralatte elettrico COMPACT con 5 modalità di funzionamento	46
<b>LT</b>	Elektrinis pientraukis COMPACT su 5 režimais	50
<b>LV</b>	Elektriskais laktators COMPACT ar 5 darba režīmiem	54
<b>NL</b>	Electrische borstkolf COMPACT met 5 bedieningsmodi	58
<b>PT</b>	Bomba tira leite eletrica COMPACT com 5 modos de funcionamento	62
<b>RO</b>	Pompă de sân electrică COMPACT cu 5 moduri de lucru	66
<b>RU</b>	Молокоотсос электрический COMPACT с 5 режимами работы	70
<b>SE</b>	Elektrisk bröstpump COMPACT med 5 driftlägen	74
<b>SK</b>	Elektrická odsávačka materského mlieka COMPACT s 5 pracovními režimami	78
<b>UA</b>	Молоковідсмоктувач електричний COMPACT з 5 режимами роботи	82

# COMPACT electric breast pump with 5 modes

## INTENDED USE

- Expressing milk from a woman's breasts
- Expressing excess milk from overfilled breasts
- Storage of excess milk to be used later

## NOTE!

- Before using the breast pump for the first time make sure you read the user manual and warnings. Keep the user manual for future reference.
- Use the breast pump in accordance with its intended use described in the user manual.
- Do not use the breast pump if it is physically damaged.
- Do not place the breast pump near devices generating a strong magnetic field.
- Do not expose the breast pump to direct sunlight.
- Keep the breast pump out of reach of children.
- Keep the pumping unit of the breast pump in a dry place and do not immerse it in water.
- Before each use, check all the elements visually for cracks, chips, tearing, stains, discolouration and signs of wear since they may affect the efficiency of the device. If the breast pump is damaged it cannot be used until the damaged parts are replaced with new ones.
- Keep the breast pump clean.

## BEFORE EACH USE

Find a quiet place and make sure that all the elements of the breast pump coming in direct contact with the food (silicone breast funnel, breast pump body, silicone membrane, silicone valve and bottle) have been sterilised and correctly assembled.

- Wash your hands thoroughly with soap and warm water before touching any element of the breast pump.
- After taking a comfortable sitting or semi-recumbent position, gently massage the breasts and heat them with a warm towel.
- Before using the breast pump make sure that all the parts have been cleaned, sterilised and dried.
- Make sure there is no liquid in the tubing. Do not switch the breast pump unless the tubing is empty.

## ADVANTAGES

- Anti-backflow protection
- The closed system of the breast pump guarantees that milk or impurities cannot enter the tubing or the pumping unit itself.

Such design protects the food and the breast pump against contamination and cross infection.

- Hygiene and safety

The Babyono breast pump is easy to clean, assemble and use. All its elements are made of non-toxic and food-safe plastic. They can be sterilised at high temperature.

- Personalised configuration

Optionally available various funnel sizes allow to ensure maximum comfort while expressing milk. A silicone or polypropylene valve with a membrane offers more options for using and cleaning the breast pump.

- Silicone funnel non-irritating to the breasts
- Gentle massaging of the areola

The soft funnel contracts inwards, gently squeezing the areola, thus increasing the efficiency of expressing milk.

- Fitted and tight funnel

The funnel non-irritating to the breasts is made of high quality silicone approved for contact with food. It is soft and properly shaped, thus perfectly fitting breasts of various shapes and sizes.

- Possibility of expressing milk in the semi-recumbent position

The funnel made of soft silicone covers the entire breast at every angle, also in the semi-recumbent position. This provides greater comfort for new mums, in particular after caesarean section.

- Unique suction mode

The Babyono breast pump offers several different modes of expressing milk that can be selected by mums according to their individual needs.

- 2 in 1 mode

A cycle of 8 massage movements followed by one suction movement.

This mode is recommended for soothing the pain of overfilled breasts and for stimulating them to express more milk in less time.

- Dual frequency mode


The mode combines the advantages of alternating high to low suction strength.

The suction cycle with a variable strength (low – break – high) effectively helps in stimulating inverted nipples.

- High frequency mode

Satisfying milk expression while stimulating the breast to stimulate the oxytocin reflex increasing the speed of let-down.

## LIST OF BREAST PUMP PARTS

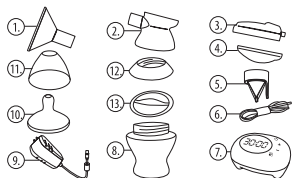1. Silicone breast funnel
2. Breast pump body
3. Pump body cap
4. Silicone membrane
5. Silicone valve
6. Silicone tubing
7. Pumping unit
8. Feeding bottle
9. Charger
10. Teat
11. Bottle cap
12. Cap
13. Bottle sealing disc

## ASSEMBLY INSTRUCTION

1. Insert the breast funnel (1) into the breast pump body (2).
2. Put the silicone membrane (4) on the breast pump body.
3. Put the pump body cap (3) over the membrane, turning it to the right until it locks.
4. Put the silicone valve (5) into the bottom of the pump body (make sure it's been done correctly).
5. Fix the feeding bottle (8).
6. Fix the silicone tubing (6) onto the splined aperture in the pump body and connect the other end to the pumping unit (7).

## USER MANUAL

- a) battery level indicator, b) clock and suction strength indicator, c) level down, d) level up, e) operating mode

- 1) Expression, 2) Double frequency, 3) High frequency
- 4) Stimulating massage, 5) Two in one

Take the correct and comfortable position. Put the funnel to the breast so that the nipple is in the funnel tube. Make sure the tubing is not bent or tangled.

Touch the ON/OFF button for 2 seconds to turn on the power. The display will start. Starting the breast pump activates the time switch set for 1 minute. Unless you start pumping the milk within that time, the pump will turn off automatically.

Touch e) symbol to select the operating mode.

Select 1) to turn on the EXPRESSION mode. The suction strength can be set in the range from 1 to 7. Individual levels differ in terms of suction strength, from low to high.

Select 2) to turn on the DUAL FREQUENCY mode. The mode combines the advantages of alternating high to low suction strength. The suction cycle with a variable strength (low – break – high) effectively helps in stimulating inverted nipples. The suction strength can be set in the range from 1 to 7.

Select 3) to turn on the HIGH FREQUENCY mode. Massage of the nipple with high frequency and low suction strength helps in stimulating the mammary gland to stimulate lactation and increase let-down.

Select 4) to turn on the STIMULATING MASSAGE mode. It is recommended to start a stimulating massage approx. 2-5 minutes before pumping the milk. The suction strength can be set in the range from 1 to 5. Each level offers a massage with a different frequency.

Select 5) to turn on the TWO IN ONE mode. The breast pump performs 8 massage cycles and 1 suction cycle. This mode is recommended for soothing the pain of overfilled breasts and for stimulating them to express more milk in less time. The suction strength can be set in the range from 1 to 7. Each level gently massages the breasts and offers different suction strength, from low to high.

Sleep mode: Unless you start pumping the milk within a minute after turning on the pump, it will turn off automatically. The default pump operating time is 30 minutes. After working for 30 minutes, the breast pump turns off automatically.

Memory mode: The breast pump automatically remembers the last used operating mode and selected suction strength.

## Warning

USE ONLY THE CHARGER SUPPLIED WITH THE BREAST PUMP.  
MAKE SURE THE CHARGER IS COMPATIBLE WITH THE POWER SOURCE.

## CHARGING THE BATTERY

**NOTE!** Before first use, the battery must be fully charged. The breast pump is equipped with a 2600mAh lithium battery, so it does not require the use of (AA type) batteries or permanent connection to power supply. It is recommended to charge the breast pump before starting to express milk.

The battery provided with the breast pump can be charged approx. 500 times. Stop charging when the battery is full in order to avoid harming the battery. In order to extend the life of the breast pump and reduce the electricity consumption by the battery it should not be charged for more than 6 hours.

## BATTERY PARAMETERS

Battery capacity	2600 mAh
Max. negative pressure	-0.04 MPa
Rated voltage	5V
Rated current	1A
Rated power	5W
Charging time	approx. 5 hours
Operating time	approx. 2.5 hours

The breast pump is equipped with a battery. To charge it, connect the pump to the charger and then to the power outlet. It will be fully charged after approx. 5 hours. During charging the indicator alternates between blue and white. When the battery is charged, the indicator turns white. The red indicator means the battery is low and needs to be recharged immediately. The battery discharge rate depends both on the mode and operating time of the breast pump and on the suction strength set. Proper operation of the built-in battery allows to extend its life. It is recommended to turn off the pumping unit of the breast pump while the battery is charging. The battery should be replaced by a specialist. Do not disassemble the built-in battery without the consent of the manufacturer.

The breast pump can be recharged at any time, without having to wait for the battery to discharge.

## SAFETY RULES

The following safety rules should be always complied with:

- In the case of inflammation of the mammary glands and inverted or flat nipples the battery-powered breast pump should be used only after consultation with a physician.
- The breast pump is intended for personal use. Sharing it by more people poses a health risk.

- Store the battery-powered breast pump out of the reach of children.
- Do not use the battery-powered breast pump when you are asleep or grow drowsy.
- Do not use the battery-powered breast pump if it is damaged.
- Do not use the battery-powered breast pump equipped with parts from other manufacturers. It may reduce its efficiency.
- Do not attach a glass feeding bottle to the battery-powered breast pump. It can get broken if falls down.
- Do not leave the battery-powered breast pump unattended when it is connected to a power source.
- Do not reach for the battery-powered breast pump if it falls into the water. In such a situation, immediately disconnect it from the power source.
- Do not expose the battery-powered breast pump to direct sunlight.
- All repairs of the breast pump should be made by an authorised service provider.

## CLEANING AND STERILISATION

Before using the battery-powered breast pump, dismantle, wash and sterilise all its components.

Improperly dried parts can interfere with the proper operation of the battery-powered breast pump. Before use, make sure all the parts are dry.

### Cleaning:

- Do not clean or sterilise the pumping unit, charger, silicone tubing and tube connector.
- Clean and sterilise all the parts coming in direct contact with the food: silicone breast funnel, breast pump body, silicone membrane, silicone valve and bottle.
- Wash the parts of the breast pump with warm water to avoid contaminating the food with bacteria.
- Do not clean with solvents or abrasives.
- Wipe the pumping unit with a cloth lightly moistened with water or water mixed with a mild soap.

### Cleaning the tubing:

- Wipe the outside of the tubing with a clean, damp cloth. Make sure that no water or other liquid gets into the tubing.
- If there is any fluid in the tubing, rinse it with clean, hot water. Then hang until completely dry. Do not insert any objects into the tubing.

## TROUBLESHOOTING

### TOO LOW OR NO SUCTION STRENGTH

- Make sure that all the parts of the breast pump are correctly assembled.
- Make sure that the membrane is properly fixed and placed flat against the breast pump body.
- Check that the silicone valve and the silicone breast funnel of the breast pump are correctly assembled.
- Make sure that the silicone breast funnel tightly adheres to the breast.

### BREAST PUMP IS WET

- Turn off the breast pump. Disconnect the charger from the power source.
- Do not put the breast pump upside down. The control panel must be facing up.
- Leave the breast pump for 24 hours in a dry, warm place.
- Do not expose the breast pump to direct sunlight.
- Do not accelerate the drying process by heating the breast pump elements.

### BREAST PUMP DOES NOT START

- Make sure that the power cord is properly connected to the breast pump.
- Make sure the power source is live.
- Contact the store where the breast pump was purchased.

### NATURAL NURSING FEEDING BOTTLE AND TEAT

The teat supplied with the breast pump is intended for feeding bottles from the Natural Nursing product family. Before each use, please check the teat by pulling it in all directions. If the teat is damaged, it should be immediately replaced with a new one. Do not enlarge the apertures in the teat. It may damage the silicone structure. For reasons of hygiene and safety, it is recommended to replace the teat every 1-2 months

### IMPORTANT FOR YOUR BABY'S HEALTH AND SAFETY

- Never leave your child unattended during feeding.
- Do not use feeding teats as soothers.
- Prolonged sucking of food may cause tooth decay.
- Always check the food temperature before feeding.
- Always assemble sterilised bottle elements with clean hands.
- The product complies with the requirements of the EN 14350 standard.

### EXPRESSED BREAST MILK STORAGE

**IMPORTANT:** Please make sure any expressed breast milk is correctly stored! Incorrectly stored breast milk loses its nutritive value and may even be harmful to your baby.

We recommend Babyono sterile bags for breast milk storing and freezing.

STORAGE PLACE	STORAGE TIME	
	FOR HEALTHY BABIES	FOR PREMATURE AND ILL BABIES
In room temperature	8 – 12 hours	1 – 4 hours
In a fridge (5° C)	3 – 4 days	24 – 48 hours
In a freezer compartment (-10° C)	2 weeks	1 week
In a freezer (-18° C)	6 – 12 months	3 – 6 months

### NOTE

- Thawed milk must not be frozen again. Any unused milk should be discarded.
- Do not mix any thawed breast milk with freshly pumped breast milk.
- You can freeze only fresh breast milk.
- Breast milk should be defrosted slowly. To do this, frozen milk should be placed in a refrigerator until it takes a liquid form.
- To warm breast milk, it is recommended to use a water bath with a temperature not exceeding 50°C.
- To preserve its nutritive value, breast milk must not be boiled, heated up directly on a cooker or in a microwave!
- Ready to serve breast milk should have the human body temperature, i.e. no more than 37°C.

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